

1) You need Medijet®, tape, bonnets (8 sizes, colour coded), prong (7 sizes) or mask (4 sizes), fixation pillow


2) Use tape to find right size of prong, mask and bonnet. Start as pictured, compare template with nares

3) Choose prong as large as possible. Connect as pictured, Keep a little distance between septum and prong

4) Choose mask as small as possible, use angle of adapter more wide

5) Place bonnet on the head, keep Velcro in the middle of the forehead, open Velcro and set bonnet as pictured and fix the generator with prong/mask at the bonnet, fix strips as pictured

6) Be sure that the circuit does no leverage and impact on the generator and place it accordingly

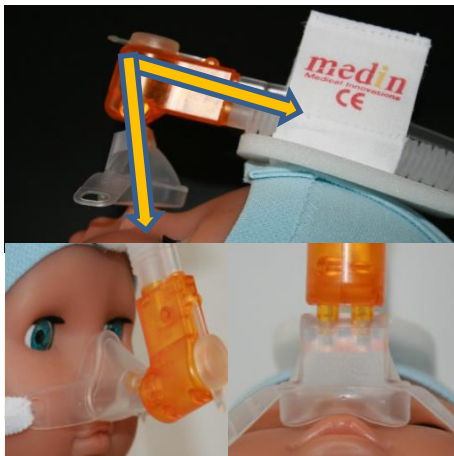

Short Application Guide Medijet® Generator



1-3) Miniflow[®] has to be fixed with the middle part of the Velcro

4+5) Reduce and avoid leverage and impact of the circuit



- How to choose prongs, Masks and bonnets see Medijet[®] instruction
- How to adjust prong/mask adapter see Medijet[®] instruction
- How to apply prongs, masks and bonnets see Medijet[®] instruction